SOPIRA® Citoject®

DE	Gebrauchsanweisung.....	2
GB	Instructions for use.....	6
FR	Mode d'emploi	10
ES	Instrucciones de uso	14
IT	Istruzioni per l'uso	18
PT	Instruções de uso.....	22
NL	Gebruiksaanwijzing	26
SE	Bruksanvisning	30
DK	Brugervejledning	34
NO	Bruksanvisning	38
FI	Käyttöohjeet	42
GR	Οδηγίες Χρήσης	46
LV	Lietošanas instrukcija.....	50
LT	Naudojimo instrukcija.....	54

KULZER

MITSUI CHEMICALS GROUP

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SOPIRA® Citoject®

– the injection system for intraligamental anaesthesia

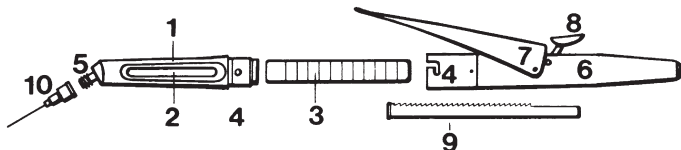
Instructions for use (GB)

Instructions for use

1. Construction of the SOPIRA Citoject syringe

The SOPIRA Citoject syringe is composed of

- barrel with viewing chamber for anaesthetic cartridge, plastic sleeve and threaded nozzle for injection needle
 - hand piece with dosing lever, resetting key and dosing plunger.
- Both parts are connected by a bayonet catch.

- | | |
|--|----------------------|
| 1 barrel | 6 hand piece |
| 2 viewing chamber for cartridge | 7 dosing lever |
| 3 plastic sleeve | 8 resetting key |
| 4 bayonet catch | 9 dosing plunger |
| 5 threaded nozzle for injection needle | 10 disposable needle |

2. Preparation

Turning the bayonet catch (4) dismantle the SOPIRA Citoject syringe.

To return plunger (9) to starting point, depress the resetting key (8) and hold syringe in an upright position (bayonet catch up).

Place anaesthetic-cartridge in the barrel and reassemble the two parts of the SOPIRA Citoject at the bayonet catch.

Depress the dosing lever (7) until slight resistance is felt.

The dosing plunger is then in contact with the anaesthetic cartridge bung and fixes the cartridge in the barrel.

Insert a disposable needle into the nozzle (5) and screw the hub onto the thread. We recommend the use of SOPIRA Carpule disposable needles in the sizes of 0.3 x 12 mm or 0.3 x 16 mm with the corresponding short needle point.

Depress the dosing lever until anaesthetic solution emerges from the needle point. The syringe is ready for use.

3. Handling the SOPIRA Citoject syringe

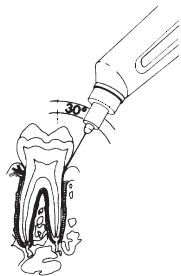
Each depression of the dosing lever applies 0,06 ml of anaesthetic.

Normally 0,15 ml–0,20 ml of anaesthetic is required for each root, i. e. 3 depressions of the lever.

Multi-rooted teeth need an appropriate dose of anaesthetic.

4. Performing an intraligamental anaesthesia with SOPIRA Citoject

Intraligamental anaesthesia provides effective and immediate anaesthesia of a single tooth.

Recommended procedure is as follows:

- In contact with the tooth, the tip of the needle is inserted 1–2 mm into the periodontal space approximately along the tooth axis.
- the needle is correctly placed in the periodontal space if a steady, uninterrupted back-pressure is noticeable
- This resistance must be maintained, to secure penetration of the solution to the apex.
- Inject slowly and uniformly.

One depression of the dosing lever should take about 7 seconds, i. e. for 0.18 ml per root = 3 depressions of the dosing lever = 21 seconds.

The onset of anaesthesia is generally obtained within 30 seconds following the injection. Hence the treatment can be started at once.

5. Cleaning and care for the SOPIRA Citoject

Please follow these instructions to maintain your SOPIRA Citoject syringe always in perfect working condition:

- After use, separate the hand-piece and barrel at bayonet catch. Remove anaesthetic protective cartridge, plastic sleeve and dosing plunger.
- **Cleaning:** All parts should be cleaned under hot running water. Important: The conventional SOPIRA Citoject syringe contains non-ferrous light metals (Aluminium) whereas only stainless steel is used in the heavier Citoject syringe. No use of acid or alkaline agents for the non-ferrous SOPIRA Citoject. For cleaning and disinfection please observe the respective manufacturer's instructions for use. Never leave the non-ferrous SOPIRA Citoject syringe immersed in disinfecting or cold sterilisation solutions beyond the manufacturer's recommended duration. Ultrasonic cleaning is possible. No cleaning in an automatic thermo-disinfector in case of the non-ferrous syringe whereas **the stainless steel SOPIRA Citoject syringe can be cleaned together with all other stainless steel instruments using any kind of common cleaning method at the dentists.**
- **Sterilisation:** Up to 135°C/275°F in an autoclave. Do not sterilise the SOPIRA Citoject whilst assembled. Plastic protective cartridge to be placed apart from metals and autoclaved up to max. 135°C/275°F. Hotair sterilisation up to 180°C/356°F is possible, but this can affect the appearance of the SOPIRA Citoject which remains in perfect working order in any case. **The stainless steel SOPIRA Citoject can be sterilised together with all other stainless steel instruments using any kind of common sterilisation method at the dentists.**
- Always ensure that components are completely dry before re-assembly. No need to oil.
- Please use needles with plastic hub (e.g. SOPIRA Carpule needles) only. A Metal hub can damage threaded nozzle of the barrel in the long term.

WARRANTY

Stringent quality control procedures ensure this surgical instrument is supplied in perfect working order. Your SOPIRA Citoject is guaranteed against defects in materials or workmanship for

ONE YEAR

from date of purchase. This guarantee is void if your SOPIRA Citoject has been improperly sterilized, poorly stored or incorrectly handled.

Making a warranty claim: Please send the SOPIRA Citoject syringe and the receipt to the dealer from whom you purchased the SOPIRA Citoject syringe.

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