# MM-EDTA Cream

## EN 19% EDTA Cream INSTRUCTION FOR USE

### Easier Filing with Lubricating Action

#### APPLICATIONS

Root canal preparation with files and reamers.  
Presence of calcification in the canal.

Too narrow canals in posterior teeth.

Smear layer removal.

#### MODE OF ACTION

Removal of inorganic substances by chelation (EDTA-Ca 2+).

Easier filling due its lubricating action.

Produces the cleanest canal wall after smear layer removal.

#### COMPONENTS

MM-EDTA Cream 7.0g (5.4ml) x 2.

Disposable tip.

Net weight : 14g.

#### DIRECTIONS FOR USE

1. Deposit a small amount of MM-EDTA Cream in the canal orifice.

2. Carefully pump into the canal with a fine file or reamer.

3. Re-apply MM-EDTA Cream when necessary or until cleaning and shaping is complete.

#### HOW TO USE ROTARY RING

If in contact with air, content could be hardened, so it is necessary to cover a cap on the needle after use.

Putting a syringe into a rotary ring is more convenient to handle & Use the rotary ring as a wrench to tighten or loosen a tip.

#### CAUTIONS

Use MM-EDTA Cream with discretion to prevent perforation.

Avoid extrusion of the solution into the apex.

Keep the syringe capped all the time.

#### STORAGE

Store at temperatures 18°C~24°C (64°F~75°F).

#### SHELF LIFE

2 years from the manufactured date.

Use By  
 Batch Code  
CE-Mark and identification number of notified body.

Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC  
 Authorized representative in the EC  
 Consult operating instructions  
 Manufacturer

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