

THE DO-IT-YOURSELF MONEY MAKER



Ufi Gel hard C

DIRECT CHAIRSIDE DENTURE HARD RELINE



Ufi Gel hard C

DIRECT HARD DENTURE RELINING

Ufi Gel hard C is a PMMA-like material for permanent direct chairside hard relines of full and partial dentures. It also allows lengthening of denture rims and adaptation of dentures, layer by layer over the years. Relines with

this new material do not differ from laboratory processed pieces in their hardness, finish or cleaning. The accurate direct impression with Ufi Gel hard C avoids loss of precision and the subsequent time-consuming corrections.



... more economical for you

- Complete chairside hard relining in one appointment
- Simple, time- and material-saving automix application from the cartridge
- No costs for impression material or laboratory work
- Better customer service (one short session, no waiting time without denture)
- Easy and fast to polish

... more comfortable for your patient

- Biocompatible with a neutral taste and smell
- Low exothermic temperature (98.6°F)
- Bubble-free application ensures a smooth surface
- Durable tight bond to the denture material
- Excellent flow properties for a precise fit
- Esthetic and color stable

POLYMERIZATION TEMPERATURE

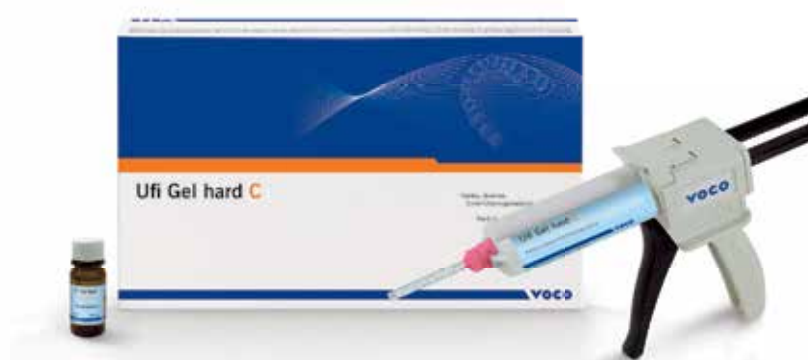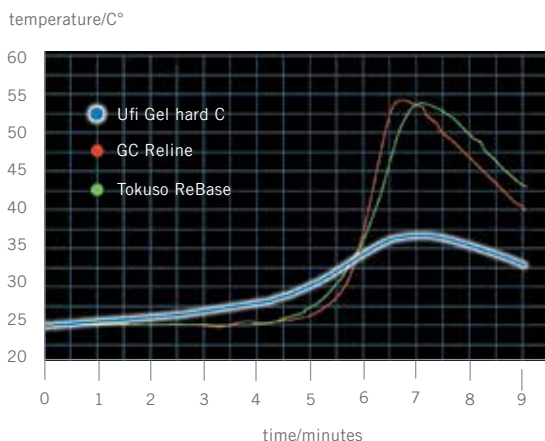

hard chairside relining in only 10 minutes ...



Ufi Gel hard C

... AS SIMPLE AS AN IMPRESSION




1
... apply adhesive ...



2
... apply Ufi Gel hard C ...



3
... functional movements ...

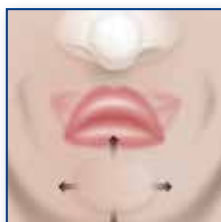

4
... remove excess ...



...then let
denture fully
cure in
the mouth.

Do the calculation and start making more money today!

PROCEDURE	DIRECT	INDIRECT
Impression Material	\$0	\$12
Ufi Gel hard C	\$20	\$0
Misc.	\$10	\$10
Lab costs	\$0	\$100 (60-240)
Office Time	\$100-\$120	1. Appoint. \$100-\$120 2. Appoint. \$100-\$120
Total	Approx. \$130-\$150*	Approx. \$382-824*

*Costs differ for every office and lab. This is an example; your calculation may be different from this result.

hard chairside relining: a win-win therapy

re-insert denture
into the mouth

remove
denture

7.0 MINUTES

curing

9.5 MINUTES

finishing, polishing

Ufi Gel Reline Products

RELINING MATERIALS WITH MORE THAN 10 YEARS OF CLINICAL SUCCESS



A-silicone based permanent soft denture relining material



Direct hard relining material in an automix cartridge for total and partial dentures, denture repairs as well as lengthening of denture rims

Presentation

Ufi Gel SC

- REF 2041 kit – cartridge 50 ml Ufi Gel SC, adhesive 10 ml, each 10 ml glazing base/catalyst, mixing tips type 8, accessories
- REF 2040 refill – cartridge 50 ml, mixing tips type 8
- REF 2029 refill 10 ml Ufi Gel SC adhesive
- REF 2047 refill each 10 ml glazing base/catalyst
- REF 2187 refill 50 mixing tips type 8 (pink)

Trimming disk:

- REF 2049 refill trimming disk, 10 pieces

Ufi hard C

- REF 2215 kit – cartridge 80 g, adhesive 10 ml, mixing tips type 8
- REF 2218 intro kit – cartridge 80 g, dispenser type 2, adhesive 10 ml, mixing tips type 8
- REF 2216 refill – cartridge 80 g, mixing tips type 8
- REF 2217 refill 10 ml adhesive
- REF 2187 refill 50 mixing tips type 8 (pink)

VOCO America Inc.
1245 Rosemont Drive
Suite 140
Indian Land · SC 29707

www.vocoamerica.com
infousa@voco.com

Toll-free phone: 1-888-658-2584
Fax: 1-888-849-3989

