DUROC CLEAR

Definition

Modified bis acrylic resin-based for making of surgical guide & bite registration

Safety notes

Warnings

Duroc Clear contains polymerizable methacrylate resins which may irritate skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.

Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.

Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.

Precautions

Wear suitable protective eyewear, mask, clothing and gloves.

Protective eyewear is recommended for patients.

Contraindication

Duroc Clear is contraindicated in patients with a history of allergic reaction to methacrylate resins or any other of the components.

Instructions for use

Preparation

Load the dispenser with the cartridge. Close the cartridge retainer on the top of the dispenser. Gently squeeze the dispenser and extrude enough material from the cartridge to ensure an even flow of both the base and the catalyst. If this does not occur, check for any plugged material and remove with an explorer. Lock a mixing tip onto the cartridge nose with a 1/4 turn clockwise. The used mixing tip must remain in place as a storage cap until the next use.

Surgical guide

1. Block off counter spaces with a thixotropic LC dam (**Notre Dam**)
2. Apply the resin on the teeth, on each side of the edentation. The layer of resin should cover at least 1 tooth distally and 2 teeth mesially. In case of terminal edentation, the resin should cover at least 4 teeth mesially.
3. Embed the "reference body" down in the resin until it is in contact with the gum.
The stem guide will be of help to "estimate" the axis of the future implant.
4. Take an Xray with the scannographic guide in place.
5. When the bi-acrylic resin is still in an elastic phase, remove the "reference body" and take an optic impression of the concerned mandibular sector.
6. The program is operating the data transmitted: prosthetical template, exportation and importation of the CAD/CAM data (3D image, implant planning process, correlation between X ray and CFAO images, choice of the implant and the implant abutment)
7. A block of the dedicated resin is drilled, following the registered datas.
8. The customized drill body is bonded to the surgical guide, using Elsodent self-etching dual-curing bonding, **Healbond Duo SE** (apply the bonding on both parts, following the IFU).
Important notice: The asymmetrical shape of the body allows one axis of insertion only.
9. Fill the free spaces with the modified bi-acrylic resin Duroc Clear, using an intra-oral tips.
Important notice: Be sure the resin will not cover the top of the drill body.
10. Trim the surgical guide to reduce it at the proper size.
11. Smooth all the angles
12. Open "windows" in the surgical guide just above the proximal teeth.
They will allow the surgeon to check if the guide is well in place, in contact with the teeth.
Important notice: do not touch the top of the drill body with burs.
13. Polish the surgical guide.

Duroc Clear can also be used to elaborate bite-registration.

Bite registration

1. If necessary, isolate the surface to be registered with a vaseline film or equivalent (in presence of new composite, for instance). Block important undercuts (pontics, embrasures) with wax, especially in case of severe gum recession.
2. Apply Duroc Clear onto dry teeth. The bite registration can be 3 to 4 cm length.
3. Ask the patient to close, to obtain a registration of the occlusion.
4. As soon as the material started its polymerization (about after 50 seconds), ask the patient to slightly open his mouth and close again. This will allow the lift and the separation of the bite registration from the dental arcade.
5. Complete polymerization occurs after two minutes.
6. Trim the bite to keep the cusp tip with a resin bur, to avoid repositioning mistakes.
7. The bite registration can be disinfected and its inhibition layer can be removed with alcohol.

Important notice

- To achieve full arch bite registration, apply Duroc Clear
- To register a complete dental arch, use Duroc Clear in segments (3-4 teeth) achieve full arch bite registration, apply Duroc Clear segment by segment of 3 to 4 teeth.
- Slightly dry teeth before registration.

Storage & shelf-life

Store at a temperature not exceeding 25°C. When stored under such conditions, Duroc Clear has a shelf-life of 24 months. Refrigerate when not in use (for example, overnight and on weekends). When cold, these materials have a stiffer consistency. For easier handling, remove from refrigerator at least 15 minutes prior to use.

Presentations & references

Duroc Clear - 50ml dual cartridge + 10 yellow mixing tips

DUCL-50

Important notice

- Do not throw away the box and the instruction for use until the product is completely finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

Disclaimer

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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