Description/Indication

Desensibilizator & liner

How does it work? & main indications

- HEMA (hydrophilic resin) seals the dentinal tubules to prevent fluid flow by combining with the proteins to produce a physiological barrier and prevent any bacterial post-colonization.
- Benzalkonium chloride eliminates the residual bacteria from the tubules. These germs are frequently responsible for post-operative hypersensitivities after fillings.
- Sodium Fluoride reduces the incidence of root surface caries, re-mineralizing the dentin, the only way to provide long-lasting desensitization.

Safety notes**Warnings**

- Healdent contains polymerizable methacrylates which may irritate skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.

Precautions

- Wear suitable protective eyewear, mask, clothing and gloves
- Protective eyewear is recommended for patients.

Contraindications

The use of HEALDENT is contraindicated if the patient is known to be allergic to any of the ingredients in HEALDENT.

Indications**Liner utilisation**

1. Prepares cavities. If the cavity is deep, apply a calcium hydroxide layer before using HEALDENT.
2. It is not necessary to have a dry dentin surface before applying HEALDENT - a slightly wet surface is acceptable.
3. After peripheric enamel and dentin are etched, apply HEALDENT on the dentin with a disposable applicator, using gentle but firm rubbing motion. Do not rinse.
4. Wait 15 seconds and gently blot dry air. Then, apply a layer of the bonding of your choice and light-cure it. (HEALDENT is compatible with all bondings).
5. Apply your composite and light-cure it.

Desensitizer utilisation**Technique 1**

1. Providing the tooth is not too sensitive, clean the cervical area with pumice and rinse area with water.
 2. Apply HEALDENT for 30 seconds with a disposable applicator, using gentle but firm rubbing motion.
 3. Lightly blot dry or let air dry. Do not rinse.
- Repeat application if required.

Technique 2

1. If area is too sensitive to clean with pumice:
2. Apply directly HEALDENT following technique 1.
3. Lightly blot dry or let air dry. Do not rinse.
4. Apply a second layer of HEALDENT.
5. If the patient is not hyper-sensitive anymore, apply a 3rd layer of HEALDENT. If patient is still hyper-sensitive, deliver a local anaesthetic and apply HEALDENT using Technique 1.

Other indications

- After tooth bleaching & whitening.
- After scaling, root planning & perio treatments.
- In extreme cases where cervical sensitivity is a continuous problem, HEALDENT can be used with any moist bonding system.

Storage & shelf-life

Do not expose the product to sun-light or warm sources. Store at ambient temperature not exceeding 25°C. When stored under such conditions, Healdent has a shelf-life of 24 months.

Important notice

- Do not throw away the box and the instruction for use until the product is completely finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

Disclaimer

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the instructions for use. Liability cannot be accepted for damages resulting from failure to observe the instructions for the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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