

# Get ready for easy composite layering

with GC Modeling Liquid & Gradia brushes



012243 - GC Modeling Liquid, Refill 6mL

001533 - GC GRADIA, Brush Round, n°1 x10

001534 - GC GRADIA, Brush Flat, n°1 x10

001535 - GC GRADIA, Brush Holder x2



900743 - GC Composite Modeling Kit

Including GC Modeling Liquid,  
Gradia Brush Flat (x5), Gradia Brush Round (x5),  
Gradia Brush Holder (x2) & Mixing Pads.

## Discover great handling and aesthetics united

with GC paste composites



Gradia Direct Anterior / Posterior / X



G-aenial Anterior / Posterior



Essentia

**GC EUROPE N.V.**  
Head Office  
Researchpark  
Haasrode-Leuven 1240  
Interleuvenlaan 33  
B-3001 Leuven  
Tel. +32.16.74.10.00  
Fax. +32.16.40.48.32  
info.gce@gc.dental  
<http://www.gceurope.com>

**GC UNITED KINGDOM Ltd.**  
Coopers Court  
Newport Pagnell  
UK-Bucks. MK16 8JS  
Tel. +44.1908.218.999  
Fax. +44.1908.218.900  
info.uk@gc.dental  
<http://uk.gceurope.com>





Apply, shape,  
smooth,  
enjoy!

Layering  
composites  
just got  
easier

with GC Modeling Liquid  
& GC brushes

**GC**

# Simplify your composite works using the brush technique

Direct composites can be tricky: they sometimes stick to your instruments, can be difficult to shape or smooth just the way you want, and at times you end up with visible margin lines...

Using a brush and a modeling resin to apply composites is a true game changer.



**Quick & easy**  
application  
and shaping

Using a **brush wetted with a modeling liquid** makes composites less sticky, resulting in **easier and faster application & shaping**.



**A smooth finish**  
for a shorter finishing  
procedure

Applying the final layer using a wetted brush also enables to **smooth it efficiently** and to achieve a very nice surface. This **simplifies the finishing procedure** and saves valuable time.



**Invisible**  
for the best aesthetic  
results

Although slightly yellowish during application, GC Modeling Liquid becomes **fully transparent after light-curing** – guaranteeing the **best aesthetic results**.

GC Modeling Liquid is the perfect wetting resin for the brush technique:



- Optimal **low viscosity**
- **Completely invisible** after light-curing
- Very **simple & time-saving** procedure
- 350 drops per bottle for **350 restorations!**
- Compatible with **all direct composites**
- Storage at **room temperature**





# Goodbye tricky composites

## Apply, shape, smooth, enjoy!



Dispense one drop of GC Modeling Liquid on a pad.



Use it quickly and protect it from the light with a cover.



Touch the drop with the brush in order to moisten it.



Remove the excesses thoroughly using a paper tissue.



Use a flat brush to create a palatal enamel shell.



A round brush can be useful to create the interproximal part.



Apply the dentin paste using the flat brush.



Model the dentin mamelons using a round brush.



Model the dentin mamelons using a round brush. Thanks to the brush, the final layer will have a smooth surface.

## Also optimal for posterior cases

A round brush helps to build the cusps and refine the design of the fissures.



The brush can be easily bent for a better posterior access.



The brush can be used to apply the material cusp by cusp.



Optimal to recreate the fissures & refine anatomy.