

PROSEPT® Wipes – The ready to use wipes for the quick, broad and residue free disinfection of surfaces.



PROSEPT® Wipes

PROSEPT® Wipes are pre-moistened, single-use wipes made of tear-resistant non-woven material. PROSEPT® Wipes are perfectly suitable for the quick and easy disinfection and cleaning of surfaces of medical devices such as examination tables, dental treatment units, medical appliances and non-invasive instruments. The carefully balanced formulation based on alcohols offers a broad effective spectrum and short contact times even in the presence of high organic contamination such as blood or saliva. The low content of volatile organic compounds (VOC) of less than 50% minimizes skin- and breathing problems, provides excellent material compatibility and results in residue-free drying. PROSEPT® Wipes are free of aldehydes and phenols and are biodegradable. The convenient refill pouches ensure an economical and efficient disinfection.

Your Benefits

- ▶ **Ready-to-use disinfectant wipes**
The use of wipes eliminates the need for spray disinfection, which can cause respiratory tract, eye and nose irritation.
- ▶ **Highly saturated wipes**
The disinfectant wipes remain moist for up to half a year in a tightly closed box.
- ▶ **Residue-free drying**
Leaves no streaks, which means additional wiping is not needed.

Effective Spectrum / Exposure Times

Bactericidal (1)	60 seconds
Antibiotic-resistant bacteria (2)	60 seconds
Tuberculocidal (3)	60 seconds
Mycobactericidal (3,4)	60 seconds
Yeasticidal (5,6)	60 seconds
Antibiotic-resistant yeasts (7)	60 seconds
Fungicidal (6)	5 minutes
Limited spectrum virucidal (8,9)	30 seconds
Bovine viral diarrhea virus (8)	15 seconds
Coronavirus (8,10)	30 seconds
Hepatitis B virus (8,9)	30 seconds
Hepatitis C virus (8,11)	15 seconds
Herpes simplex virus type 1 (8)	15 seconds
Human immunodeficiency virus (8,9)	30 seconds
Influenza A virus H3N2 (8)	15 seconds
Vaccinia virus strain Elstree (8)	30 seconds
Virucidal (12,13)	60 seconds
Adenovirus type 5 (12)	60 seconds
Bovine enterovirus type 1 (14)	60 seconds
Norovirus MNV (12)	60 seconds
Poliovirus type 1 LSc-2ab (12)	60 seconds
Rotavirus strain Wa (12)	30 seconds

(1) EN 1040, EN 1276, EN 1656, EN 13727, EN 13697*, EN 14349*, EN 14561*, (2) A. baumannii, ESBL, K. rhizophila, MRSA, VRE, (3) EN 14348, EN 14563*, (4) EN 14204, (5) EN 1275, EN 1650, (6) EN 1657, EN 13624, EN 13697*, EN 14562*, (7) CaDR, (8) RKI/DVV guidelines, (9) active against all enveloped viruses incl. HBV and HIV, (10) BCoV surrogate virus, (11) BVDV surrogate virus, (12) EN 14476, (13) active against all enveloped and non-enveloped viruses, (14) EN 14675, complies with EN 13610, * carrier test

Delivery Forms

OD-041079	Refill pouch with 120 disinfectant wipes (14.5 x 20 cm), classic scent
OD-041065	Wipes tub of 120 disinfectant wipes (14.5 x 20 cm), classic scent
OD-041084	Refill pouch with 120 disinfectant wipes (14.5 x 20 cm), lemon scent
OD-041068	Wipes tub of 120 disinfectant wipes (14.5 x 20 cm), lemon scent

Easy to Use

- ▶ For your protection, wear disposable gloves.
- ▶ Before use, cut open the refill pouch. Pull the first wipe at the heart of the roll through the star-shaped opening in the lid.
- ▶ Close the wipes box firmly after each use.
- ▶ Use the wipe to completely moisten the surface or medical device to be disinfected.
- ▶ Remoisten PROSEPT® Wipes with PROSEPT® Spray if required.
- ▶ Follow contact times.
- ▶ If necessary, dry the surface with a clean single-use cloth.
- ▶ Before first time use, test material compatibility by applying the product to a small inconspicuous area.
- ▶ Use only on alcohol compatible materials.

Time of Application

- ▶ After every treatment, disinfect all surfaces within a radius of 2 m from the dental chair, including outer surfaces of hand-pieces and contra-angles.
- ▶ In case of visible contamination with blood or other potentially infectious material, immediately clean and disinfect all contaminated areas or objects.
- ▶ Disinfect all surfaces in the treatment room once daily.

Important Advice

- ▶ Close the wipes box firmly after each use.
- ▶ Use wipe instead of spray disinfection whenever possible to prevent aerosol formation.
- ▶ Use only on alcohol compatible materials.
- ▶ Before first time use, test material compatibility by applying the product to a small inconspicuous area.
- ▶ Single-use cloths made of recycled paper often harbour an elevated number of germs and should therefore be avoided.

Tips

- ▶ Keep the treatment room as tidy as possible to facilitate the disinfection.
- ▶ Use PROSEPT® Spray to remoisten the wipes if necessary.

Relevant Physical and Chemical Properties

Color:	Colourless
Physical state:	Liquid absorbed on fleece
Scent:	Of alcohol
pH-value:	Neutral
pH-value in aqueous solution:	Not applicable (ready-to-use solution).
Flammability:	The product is flammable
Relative density (g/cm ³):	0.9

Composition of the moisturising solution in 100 g

31.3 g ethanol, 16.7 g 2-propanol

Shelf Life

Unopened container:	36 months
Unused working solution:	Not applicable.
Working solution in use:	Not applicable.

The shelf life of the product is subject to correct storage and transportation conditions and the use of original containers. Keep containers tightly closed.

Warning

H228 Flammable solid. H319 Causes serious eye irritation. P210 Keep away from heat/sparks/open flames/hot surfaces. No smoking. P233 Keep container tightly closed. P280.2 Wear protective gloves and eye protection. P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P337+P313 If eye irritation persists: Get medical advice/attention. P501.1 Dispose of contents/container in accordance with relevant regulations.

Your Local Distributor



Hygiene360 AG
Büelstrasse 17
8330 Pfäffikon ZH
Switzerland
Telephone: +41(0)44 226 44 99
Fax: +41(0)44 226 44 66
info@hygiene360.com
www.hygiene360.com