


THE SOLUTION: MAXCEM ELITE™ CHROMA

THE FIRST CEMENT THAT TELLS YOU WHEN IT'S TIME TO CLEAN UP.



The first self-etch/self-adhesive resin cement offering a **Color Cleanup Indicator**, Maxcem Elite Chroma is the smartest cement available on the market today.

The pink color fades at the gel state, telling you the optimal window of time to clean up excess cement.

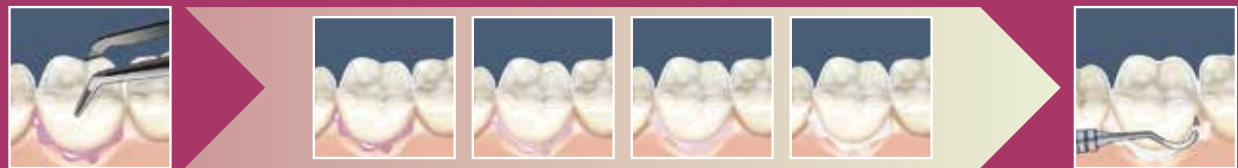

Seat the restoration and hold it with moderate pressure. Allow the cement to flow from all the margins.

Cement colour will change from pink to white as it cures. For yellow cement shade, colour will change from pink to ivory.

Once pink colour is diminished, remove excess cement while maintaining pressure on the restoration.

Maxcem Elite Chroma forms a strong and durable bond with a wide variety of dental substrates and is compatible with all indirect restorations, including:

ANTERIOR OR POSTERIOR CERAMICS

PFMS

METAL RESTORATIONS

CAD/CAM MATERIALS

DIFFERENT SOLUTIONS, THE SAME QUALITY

MAXCEM ELITE™ CHROMA

Self-Adhesive Resin Cement with Colour Cleanup Indicator

Pioneering technology

Smart cement that dispenses pink and changes to the cement shade.

Colour Cleanup Indicator shows when to clean up excess cement.

Pink disappears after both self-cure and tack cure for maximum flexibility.

Best-in-class adhesion when compared to other cements.¹

Self-etch, self-adhesive resin compatible with all indirect restorations.

280% radiopacity for easy identification on x-rays.

One-Peel™ cleanup, easy storage, no refrigeration needed.



AVAILABLE SHADES:
Clear, White, Yellow

MAXCEM ELITE™

Self-Adhesive Resin Cement

Easy and versatile

Self-etch, self-adhesive resin cement for indirect restorations is simplicity refined.

Up to 90% higher bond strength¹ than the previous formula.

Predictable handling and exceptional ease of use.

Best-in-class adhesion when compared to other leading self-adhesive cements.

One-Peel™ cleanup, no hand mixing, and no refrigeration required.

Intraoral and root-canal tips for dispensing into hard to reach places.



AVAILABLE SHADES:
Clear, White,
White Opaque, Yellow

NX3

Universal Adhesive Resin Cement

Aesthetics

Indicated for final cementation of all indirect applications.

Outstanding handling characteristics.

Tack cure capability and easy cleanup.

Available in five shades.

Try-in gels match both light-cure and dual-cure cement.

Universal. Can be used with a total etch, selective etch, or self-etch bonding system.



AVAILABLE SHADES:
Clear, White, Bleach,
White Opaque, Yellow

Maxcem Elite Chroma - Standard Kit

CONTENTS: 4x 5g Syringes (2x Clear, 1x White, 1x Yellow); 16x Automix Tips (Regular); 8x Automix Tips (Wide); 8x Intraoral Tips; 8x Root Canal Tips; Technique Guide.

Part No.	Description
36371	Maxcem Elite Chroma - Standard Kit

Maxcem Elite Chroma - Refills

CONTENTS: 2x 5g Syringes; 8x Automix Tips (Regular); 8x Automix Tips (Wide); 8x Intraoral Tips; Technique Guide.

Part No.	Description
36372	Maxcem Elite Chroma - Clear
36373	Maxcem Elite Chroma - White
36374	Maxcem Elite Chroma - Yellow

References

1. Korsch M, Robra B, Walther W. Cement-associated signs of inflammation: retrospective analysis of the effect of excess cement on peri-implant tissue. Int J Prosthodont. 2015;28(1):11-18. 2. Wilson T. The positive relationship between excess cement and peri-implant disease: a prospective clinical endoscopic study. J Periodontol. 2009;80(9):1388-1392. 3. Korsch M, Walther W. Peri-implantitis associated with type of cement: a retrospective analysis of different types of cement and their clinical correlation to the pre-implant tissue. Clin Implant Dent Relat Res. 2015;17(suppl 2):e434-e443. 4. Brown W, et al. Bond strength of several self-adhesive cements to enamel and dentin. IADR. 2015.

Maxcem Elite Chroma - Accessories

Part No.	Description
36375	40x Automix Tips (Regular)
36376	8x Intraoral Tips; 8x Automix Tips (Wide)
36377	8x Root Canal Tips; 8x Automix Tips (Wide)

Maxcem Elite™ Chroma

Self-Etch / Self-Adhesive Resin Cement with Cleanup Indicator



THE ONLY CEMENT
THAT TELLS YOU
WHEN IT'S TIME
TO CLEAN UP

CLEANUP: YOU NEVER THINK ABOUT IT UNTIL SOMETHING GOES **WRONG**.

Cleanup is one aspect of your practice that's rarely on your mind. But when timing is everything, missing the window of opportunity can threaten your restoration.

WHAT CAN AWAIT YOU?¹⁻³



CROWN DEBONDING



SOFT TISSUE INFLAMMATION



GUM RECESSION



CRESTAL BONE LOSS



PERI-IMPLANTITIS

WHEN THE PINK DISAPPEARS, YOU'RE READY TO CLEAN UP

COLOUR CLEANUP INDICATOR

Cement is pink when dispensed and gradually fades to the cement shade, indicating the optimal window of time to clean up excess cement



HIGH BOND STRENGTH

- Reliable and predictable⁴
- Best in class adhesion compared to other leading self-adhesive cements⁴
- 90% higher bond strength on dentin⁴
- 65% higher bond strength on enamel⁴
- Forms a strong and durable bond with various dental substrates⁴

Data on file. Calibra Univy, G-Cem, Panavia SA and Unicem 2, are the trademarks or registered trademarks of their respective owners.




USE WITH DESIRED SUBSTRATES

- Zirconia
- Gold
- Rexillum
- Lithium disilicate
- Porcelain
- Composite
- Titanium




AUTOMIX DELIVERY SYSTEM

- Dual barrel, automix syringe system
- Comes with single use automix tips
- Optional curved tips deliver desired amount of cement into restoration and/or tooth prep

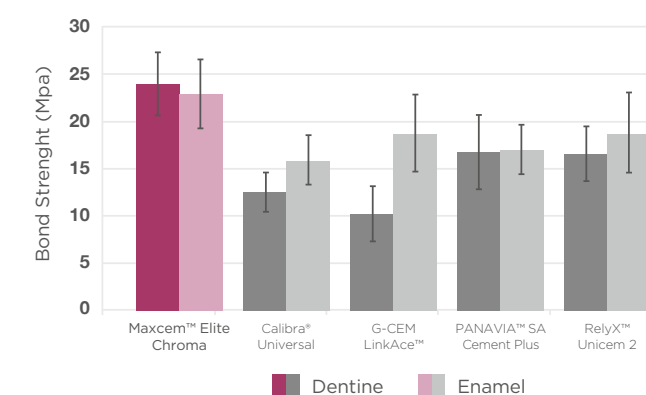

LONG TERM COLOUR STABILITY

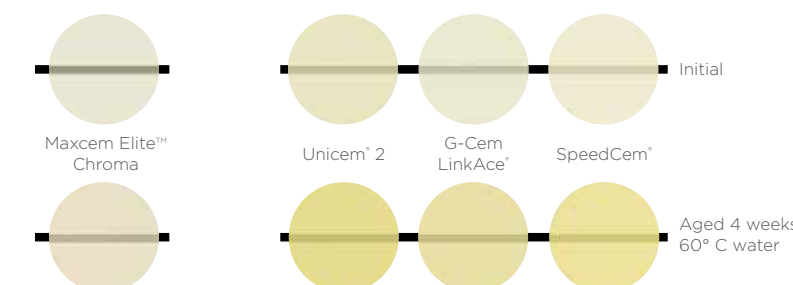

DUAL-CURE

Pink disappears after both self-cure and light cure, allowing maximum flexibility



280% RADIOCAPACITY

Outstanding radiopacity allows easy detection on a radiograph



ONE-PEEL™ CLEANUP

- Cement is easy to remove in one peel
- Reduces risk of leaving excess cement



EASY STORAGE

No need to refrigerate