Kerr™

Temp-Bond™

Temp-Bond™ NE

A Guide
for Using Kerr
Temporary Cements

English - For dental use only

Directions for use

PRODUCT BRAND NAME

Temp-Bond™ – Zinc Oxide/Eugenol Cement

Temp-Bond™ NE – Zinc Oxide/Non-Eugenol Cement

DEVICE DESCRIPTION

Temp-Bond temporary cement product series consists of Temp-Bond and Temp-Bond NE. Each is suitable for cementing temporary crowns, bridges, inlays, onlays and splints. Each has adequate flow to permit the restoration to be completely seated. They are designed to withstand the stresses of mastication and permit removal of the restoration when this is desired. Temp-Bond and Temp-Bond NE are available in traditional tubes, Unidose™ single-use foil pouches, and automix dual barrel syringes.

INDICATIONS FOR USE

Temp-Bond and Temp-Bond NE (non-eugenol) are temporary cements used in conjunction with temporary restorations such as crowns, bridges, inlays, onlays and splints.

CONTRAINDICATIONS

- Temp-Bond contains eugenol and should not be used on patients with a known allergy to eugenol.
- Temp-Bond NE may contain trace amounts of eugenol.

WARNINGS

- These products shall not be placed in contact with pulp tissue.
- Direct exposure of these products may harm the pulp.
- Avoid prolonged contact with skin, eyes, and soft tissue.
- Wash thoroughly with water after contact.
- Automix tips are intended for single patient use only to prevent cross-contamination between patients.
- Unidose is intended for single patient use only.

PRECAUTIONS

- Store at ambient temperature
- Refer to outer package for expiration date; do not use after expiry date

ADVERSE REACTIONS

None Known

INSTRUCTIONS

Work Time and Set Time: At the ideal mixing conditions of ambient temperature, the following work time and setting time from start of mixing are obtained:

Automix Syringes

	Working Time (mm:ss)	Set Time (mm:ss)
Temp-Bond	≥ 1:00	≤ 7:00
Temp-Bond NE	≥ 1:00	≤ 7:00

Tubes

	Working Time (mm:ss)	Set Time (mm:ss)
Temp-Bond	≥ 1:30	≤ 7:00
Temp-Bond NE	≥ 1:30	≤ 7:00

These times may vary based on storage conditions, temperature, humidity, and age of product.

Mixing:

Tube: Extrude equal lengths of Base and Accelerator onto the mixing pad provided. The length to be extruded will depend on the size and type of restoration to be cemented. **Replace caps tightly on tubes after use. Do not invert caps on tubes because this may cause premature hardening of the product.** Thoroughly mix the pastes for approximately 30 seconds.

Unidose: Unidose is for convenience and has the appropriate amount of base/catalyst for a restoration. Cut along dotted line with scissors, dispense contents of pouch onto mixing pad.

Thoroughly mix the pastes for approximately 30 seconds.

Syringe: Remove cap from syringe. Always bleed syringe upon the initial use and prior to attaching an automix tip. Place automix tip onto syringe. Turn automix tip 90 degrees to lock in place. The material is now ready to be dispensed directly onto the temporary. No hand mixing is necessary.

Preparation and application

Dry the prepared teeth and the surface of the restoration. Apply a thin layer of the mixed cement to the internal surfaces of the temporary restoration. Firmly seat the restoration in the mouth. After the material has set remove excess material with a scaler or other instrument.

NOTES

- A softer cement can be obtained by adding up to 25%, by volume, of white petrolatum to Temp-Bond and Temp-Bond NE.

INFORMATION REQUIRED BY ISO 3107

- Classification for Temp-Bond and Temp-Bond NE is Type I.
- For Temp-Bond and Temp-Bond NE, the volume of product in tubes is 13 to 15ml and the volume of product in Unidose pouches is 0.5

to 0.6ml. Syringes are 5ml.

- The mass ratio of product to use is:
- Temp-Bond, 3.4:1.0 (base:accelerator)
- Temp-Bond NE, 3.5:1.0 (base:accelerator)

"Temp-Bond" and "Unidose" are trademarks owned by Kerr Corporation.

"Temp-Bond" and "Unidose" are registered with the U.S. PTO and other countries.

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KerrHawe S.A.